

MAX-Ability Inc.

Pressalit Care Model 85 Series Changing Table / Shower Stretcher Product User Manual

Folding the table: The table has built-in balance cylinders for easy folding to the wall when not in use, and unfolding from the wall for changing and showering. Use two hands to fold and unfold the table.

Height adjustment (height adjustable models only): Use the hand held remote control push buttons to adjust the table height to accommodate occupant transfer and caregiver working height. The table stops when it reaches the lowest height (12") and maximum height (34"). The table may be occupied during adjustment. When not in use the hand control should be stored in its wall bracket. Do not store the hand control on or near the table folding mechanism.

Head and back support: The end pad with the built-in adjustment hardware may be placed at either end of the table. The adjustment bracket adjusts to accommodate head and back recline angles.

Safety side rail (optional): When not in use the safety rail is located under the table. Use two hands to pull down on the rail to disengage from the storage position. Swivel the rail upward and it will automatically lock into the vertical/upright position. To unlock the safety rail and move to the storage position, locate the two white spring-loaded lock knobs under the table. Using two hands pull the knobs toward each other and the rail will unlock. Swivel the rail under the table and push firmly into the table to hold the safety rail for storage.

Do not leave anyone unattended while they are occupying the table.

Cleaning the product: All components are water compatible. To remove the blue foam pads for cleaning, lift up on the end pad handle located at each end of the table and pull up to disengage the pad from the table; the middle pad will then be accessible for removal. When the pads are removed the water receiver tray is exposed for cleaning. Replace the pads by pushing them firmly into place. The pad handles should be at the head and foot of the table.

Most general purpose cleaners and disinfectants are compatible with the product. Do not use harsh brushes, abrasives or bleach products. Cleaning is recommended between each use.

General Maintenance: Preventative maintenance is recommended at six month intervals.

1. Check the table folding operation and two pneumatic cylinders (located under the table and behind the gray trim strips) for easy smooth quiet folding. Place a small amount of white lithium grease at the two hinge points located at the wall mounting bracket.
2. Height adjustable models: Height adjustment should be smooth and quiet. No lubrication or adjustment is necessary.
3. Contact MAX-Ability for assistance: 707-575-5558 or www.max-ability.com.