

MAX-Ability Inc.
800-577-1555

Installation Notes

Pressalit Care 85 Series

Changing Table / Shower Stretcher / Dressing Bench:

These notes are in addition to the installation drawings (see www.max-ability.com):

Fixed Height Models Only:

The changing table height specification (34") may be lowered to accommodate lifting devices and caregiver requirements. It is recommended that the working height not be lower than 28". For use as a 'dressing bench' ADA code requires a 17"-19" table height to accommodate independent wheelchair transfers.

All Models:

The table component is installed after the mounting hardware is anchored.

Retrofit installation: New anchor holes may be added to the mounting hardware to match existing wall studs.

The product must be anchored to solid wood, masonry or concrete.

Changing Table Users: In a remodel or new construction project a hand held shower and wall drain or floor drain outlet may be considered to accommodate shower, hygiene and disinfecting requirements. The product includes an integrated water receiver tray and flexible drain hose assembly (see drawing).

Height Adjustable Models:

Extend the leveling feet ½" prior to anchoring the hardware.

The horizontal operating frame must clear the power plug during height adjustment.

Electrical requirements: A 120v wall outlet located per the product installation drawing or within range of the 8' power cord.

The product internal electric power is 24v and 1amp.

A GFI outlet is required if using the product with a shower.

An integrated transformer is included with an 8' grounded power cord and plug.