

Linido Changing Table / Shower Stretcher User Manual

For mounting instructions see the installation notes at max-ability.com

The product is durable and reliable. When mounted correctly the weight capacity is 300lbs. The table may be folded up or down to the wall for storage. When folded into the up position it is safely held in place by the wall mounted holding device included with the product.

To use the table for showering or changing, the two hinged leg supports are unfolded. **Make sure both leg supports are pushed into the black hinges after being folded out. This locks the leg supports into position for product safety and stability.** Both legs should be touching the floor.

To return the table back to the wall for storage, make sure both leg supports are pulled out and unlocked from the black hinges before folding. **Failing to unlock the leg supports before folded back to the table may result in the hinge assembly being forced off of the metal frame.** After the legs are folded into the table a clip is provided to hold the legs in the folded position.

Cleaning: It is recommended that the product be cleaned after each use. Use a cloth or soft brush with soap and water, multipurpose liquid cleaner or a hospital grade cleaner/disinfectant (read precautions). Use a light cleanser for stains or scuffs. Bleach and harsh cleaners are not recommended. The product is water friendly for showering and cleaning/rinsing.

Maintenance: Inspect the wall anchorage and hinged parts every three months. Report loose or missing hardware. Lubrication is not necessary.

For product support call MAX-Ability: 1-800-577-1555